

# Insta-TEST<sup>®</sup>

## PRO

### PHOSPHATE

#### HIGH RANGE

3000-12000 ppb

25 Pool & Spa  
Test Strips

 **LaMotte**

Chester town, MD 21620  
[www.lamotte.com/insta](http://www.lamotte.com/insta)

**Phosphate**

High Range

Code 3040-G ppb



5x



3000

6000

12000

1. Fill test tube (0106) to 10 mL line.

2. Remove one test strip from bottle. Cap bottle tightly after removing strip. Gently bend strip in half (do not fold) with pads facing inward. Place strip inside test tube cap (as shown).

3. Cap test tube and invert slowly 5 times (invert the tube slowly to allow the bubble to go from the top to bottom and bottom to top).

4. Remove cap and test strip.

5. Hold test tube against white area of color chart (as shown) and compare to color chart.

#### TIPS

Cap bottle tightly after removing strip.

Keep wet fingers out of the bottle.

Store in cool, dry place.

For best results, read in natural light.



Drop holding box and dotted tolerance line